ORGENTEC Diagnostika GmbH

Carl-Zeiss-Straße 49-51

55129 Mainz - Germany

Phone: +49 (0) 61 31 / 92 58-0

Fax: +49 (0) 61 31 / 92 58-58

Internet: www.orgentec.com

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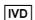
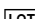
2014-06

ORG 204M Anti-dsDNA IgM

KRÁTKÝ POPIS

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REFERENCE

- Alba P, Bento L, Cuadrado MJ, Karim Y, Tungekar MF, Abbs I et al. Anti-dsDNA, anti-Sm antibodies, and the lupus anticoagulant: significant factors associated with lupus nephritis. *Ann Rheum Dis* 2003; 62(6):556-560.
- Antico A, Platzgummer S, Bassetti D, Bizzaro N, Tozzoli R, Villalta D. Diagnosing systemic lupus erythematosus: new-generation immunoassays for measurement of anti-dsDNA antibodies are an effective alternative to the Farr technique and the Crithidia luciliae immunofluorescence test. *Lupus* 2010; 19(8):906-912.
- Brouwer R, Hengstman GJ, Vree EW, Ehrfeld H, Bozic B, Ghirardello A et al. Autoantibody profiles in the sera of European patients with myositis. *Ann Rheum Dis* 2001; 60(2):116-123.
- Castro C, Gourley M. Diagnostic testing and interpretation of tests for autoimmunity. *J Allergy Clin Immunol* 2010; 125(2 Suppl 2):S238-S247.
- Defendenti C, Atzeni F, Spina MF, Grosso S, Cereda A, Guercilena G et al. Clinical and laboratory aspects of Ro/SSA-52 autoantibodies. *Autoimmun Rev* 2011; 10(3):150-154.
- Eriksson C, Kokkonen H, Johansson M, Hallmans G, Wadell G, Rantapaa-Dahlqvist S. Autoantibodies predate the onset of Systemic Lupus Erythematosus in northern Sweden. *Arthritis Research & Therapy* 2011; 13(1):R30.
- Haugbro K, Nossent JC, Winkler T, Figenschau Y, Rekvig OP. Anti-dsDNA antibodies and disease classification in antinuclear antibody positive patients: the role of analytical diversity. *Ann Rheum Dis JID* - 0372355 2004; 63(4):386-394.
- Ippolito A, Wallace DJ, Gladman D, Fortin PR, Urowitz M, Werth V et al. Autoantibodies in systemic lupus erythematosus: comparison of historical and current assessment of seropositivity. *Lupus* 2011; 20(3):250-255.
- Isenberg DA, Manson JJ, Ehrenstein MR, Rahman A. Fifty years of anti-ds DNA antibodies: are we approaching journey's end? *Rheumatology (Oxford)* 2007; 46(7):1052-1056.
- Kattah NH, Kattah MG, Utz PJ. The U1-snRNP complex: structural properties relating to autoimmune pathogenesis in rheumatic diseases. *Immunol Rev* 2010; 233(1):126-145.
- Kumar Y, Bhatia A, Minz RW. Antinuclear antibodies and their detection methods in diagnosis of connective tissue diseases: a journey revisited. *Diagn Pathol* 2009; 4:1.
- Meroni PL, Schur PH. ANA screening: an old test with new recommendations. *Ann Rheum Dis* 2010; 69:1420-1422.
- Petri M, Magder L. Classification criteria for systemic lupus erythematosus: a review. *Lupus* 2004; 13(11):829-837.
- Poole BD, Schneider RI, Guthridge JM, Velte CA, Reichlin M, Harley JB et al. Early targets of nuclear RNP humoral autoimmunity in human systemic lupus erythematosus. *Arthritis Rheum* 2009; 60(3):848-859.
- Putova I, Dostal C, Becvar R. Prevalence of antinucleosome antibodies by enzyme-linked immunosorbent assays in patients with systemic lupus erythematosus and other autoimmune systemic diseases. *Ann N Y Acad Sci* 2007; 1109:275-286.
- Reveille JD. Predictive value of autoantibodies for activity of systemic lupus erythematosus. *Lupus JID* - 9204265 2004; 13(5):290-297.
- Simon JA, Cabiedes J, Ortiz E, Alcocer-Varela J, Sanchez-Guerrero J. Anti-nucleosome antibodies in patients with systemic lupus erythematosus of recent onset. Potential utility as a diagnostic tool and disease activity marker. *Rheumatology (Oxford)* 2004; 43(2):220-224.
- Sinclair D, Saas M, Williams D, Hart M, Goswami R. Can an ELISA replace immunofluorescence for the detection of anti-nuclear antibodies?--The routine use of anti-nuclear antibody screening ELISAs. *Clin Lab* 2007; 53(3-4):183-191.
- Tozzoli R, Bizzaro N, Tonutti E, Villalta D, Bassetti D, Manoni F et al. Guidelines for the laboratory use of autoantibody tests in the diagnosis and monitoring of autoimmune rheumatic diseases. *Am J Clin Pathol* 2002; 117(2):316-324.
- Maidhof W., Hillas O. Lupus: an overview of the disease and management options. *P T* 2012; 37(4):240-9.

21. Hahn BH, McMahon MA, Wilkinson A, Wallace WD, Daikh DI, Fitzgerald JD et al. American College of Rheumatology guidelines for screening, treatment, and management of lupus nephritis. Arthritis Care Res (Hoboken) 2012; 64(6):797-808.